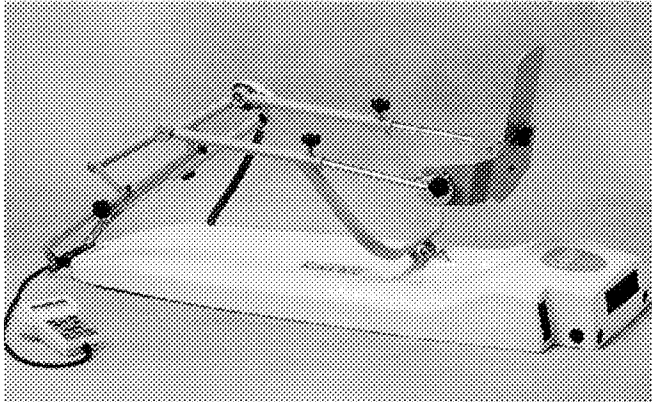
# Smith & Nephew's Kinetec® CPM machines— the widest selection in the industry.

## Optima™ Knee CPM Machine

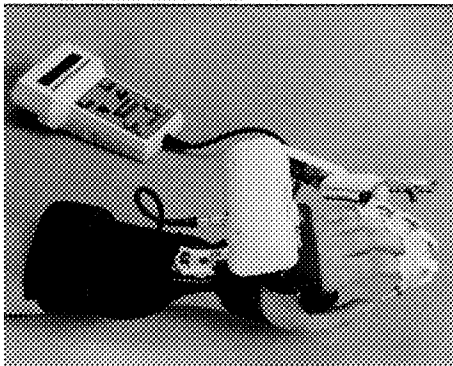
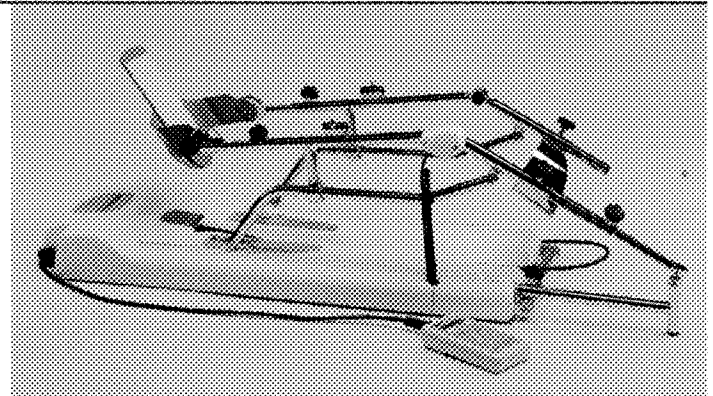
- Easy-to-use digital hand control.
- New leg carriage designed for anatomical motion.
- Computer entirely in the hand control.
- Screw drive transmission provides reliable motion.
- Programmable muscle stim capability.
- Timer provides easily programmable length of therapy.

|                                                                  |             |
|------------------------------------------------------------------|-------------|
| Optima™ Knee CPM Machine                                         | 4621-001601 |
| Replacement patient pad kit                                      | 5315-0067   |
| Adjust-a-Cinch pad system                                        | 5315-0111   |
| Premium quilted patient pad kit                                  | 5315-0110   |
| Adjust-a-Cinch straps (pkg. of 5)                                | 5315-0114   |
| Bed clamp (for use with Optima™ and Performa™ Knee CPM machines) | 4670-015724 |

## Performa™ Anatomical Knee CPM Machine

Provides anatomical motion, ideal for sports injuries.  
Computer entirely in easy-to-use waterproof hand control.  
Adult and pediatric capabilities.  
Screw drive transmission provides reliable motion.  
Programmable muscle stim capability.  
Timer provides easily programmable length of therapy.

|                                                                  |             |
|------------------------------------------------------------------|-------------|
| Optima™ Knee CPM Machine                                         | 4621-001801 |
| Replacement patient pad kit                                      | 5315-0092   |
| Adjust-a-Cinch pad system                                        | 5315-0113   |
| Premium quilted patient pad kit                                  | 5315-0112   |
| Adjust-a-Cinch straps (pkg. of 5)                                | 5315-0114   |
| Bed clamp (for use with Optima™ and Performa™ Knee CPM machines) | 4670-015724 |

## 8091™ Portable Hand CPM Machine with Soft Splint

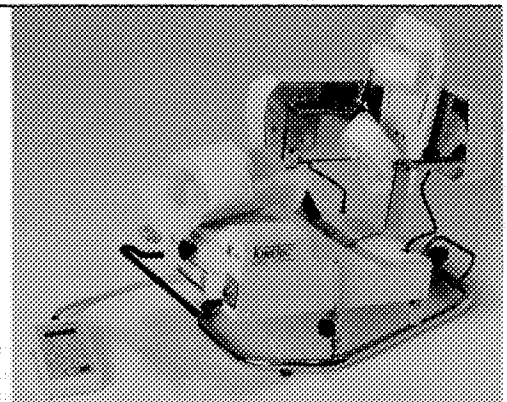
- Comfortable soft splint helps improve patient compliance.
- Splint fits either hand to reduce inventory.
- Adjustable for ulnar or radial wrist deviation.
- Lower motor mount brings drive arm into alignment with patient's joint.
- Quick set-up allows patient to receive comprehensive motion therapy quickly for a wide variety of indications.
- Skid-resistant finger wraps can be applied and removed quickly and don't leave a sticky residue.

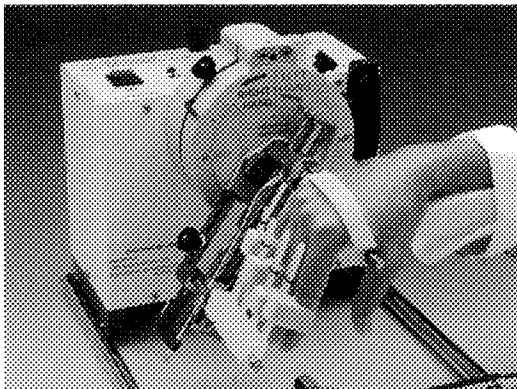
|                                                                                                                           |                     |
|---------------------------------------------------------------------------------------------------------------------------|---------------------|
| 8091™ Portable Hand CPM Machine with Soft Splint                                                                          | 4621-809101         |
|                                                                                                                           | Small Large         |
| Universal bilateral soft splint                                                                                           | 5322-0068 5322-0069 |
| Cover for soft splint                                                                                                     | 5322-0070 5322-0071 |
| Other accessories available: gloves, finger wraps, finger slides, MCP blocking system, lumbrical bill system, and more... |                     |

## 90™ Ankle CPM Machine

Three-motor axis design provides anatomically correct motion to the ankle and subtalar joint.  
Can program up to 10 individual therapy programs.  
Three modes of operation: ankle, subtalar, and combined ankle and subtalar.  
Performs reverse on load for patient safety.

|                             |             |
|-----------------------------|-------------|
| 90™ Ankle CPM Machine       | 4621-004901 |
| Replacement patient pad kit | 5315-0107   |

### 8080™ Hand and Wrist CPM Machine

- Provides post-operative range-of-motion exercise to help prevent joint stiffness.
- Allows the patient to form a true composite fist.
- Provides pronation/supination of the wrist and flexion/extension for fingers, thumb and wrist.

8080™ Hand and Wrist CPM Machine  
Replacement patient pad kit

4621-002801  
5315-1909

### 81™ Shoulder CPM Machine

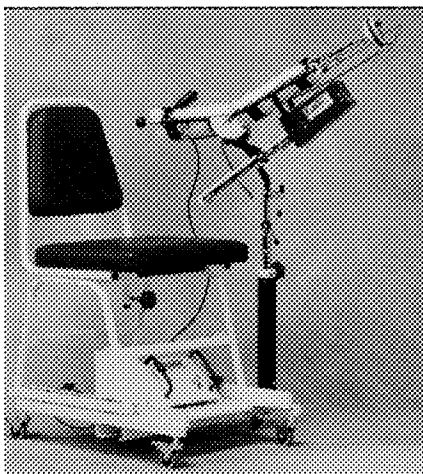
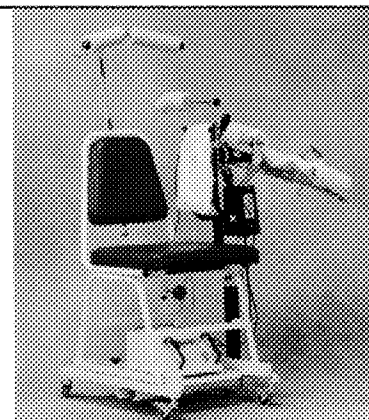
Provides post-operative range-of-motion exercise that can help prevent joint stiffness, soft tissue contractures and muscle atrophy.

Provides active rehabilitation of the rotator muscles and passive abduction of the shoulder with or without external rotation.

Can be used with the patient in bed or sitting on a chair.

31™ Shoulder CPM Machine with Base Stand  
Replacement patient pad kit  
M adjustable chair

4621-002901  
5315-0085  
4670-010617

### 6080™ Elbow CPM Machine

- Anatomically correct passive motion of the elbow joint with or without synchronized pronation/supination of the forearm.
- Conveniently located goniometer allows patient or therapist to quickly see range-of-motion progress.
- Automatic reverse on load prevents excessive force from being applied to the joint.
- Machine can be used with the patient in bed or sitting on a chair.

6080™ Elbow CPM Machine with Base Stand  
Replacement patient pad kit  
CPM adjustable chair

4621-000801  
5315-0082  
4670-010617

### 81™ Hip CPM Machine

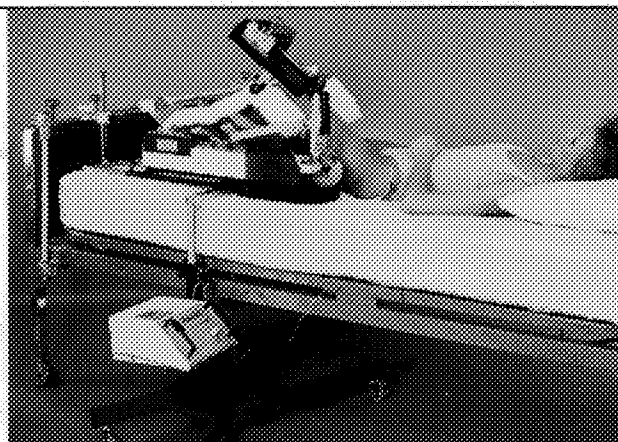
Provides optimum range of motion by allowing and controlling flexion, extension and abduction.

Shock-absorbing feature transfers the weight of the leg off the hip joint.

Can be wheeled away from bed when not in use.


31™ Hip CPM Machine with Base Stand  
Replacement pad kit

4621-001901  
5315-0088


## Smith+Nephew

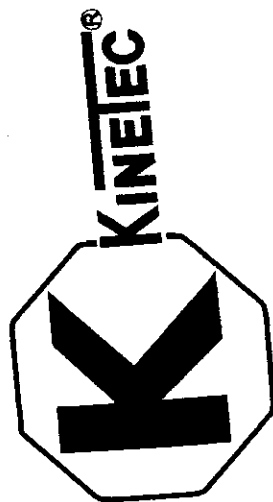Leadership in Worldwide Healthcare

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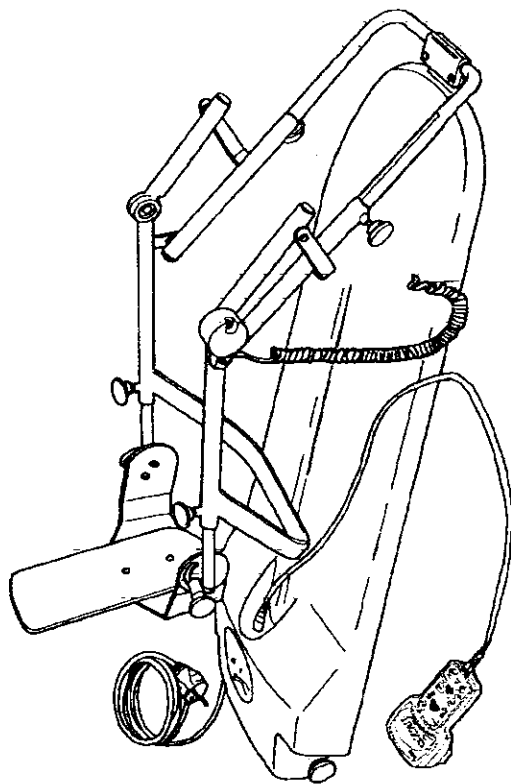
**Smith & Nephew**  
*Leadership in Worldwide Healthcare*

**Smith & Nephew Kinetec S.A. - Z.I. de Tournes-Cliron - B.P. 1109 - F-08090 TOURNES**  
**Tél. : 03.24.52.91.21 - Fax : 03.24.52.90.34**

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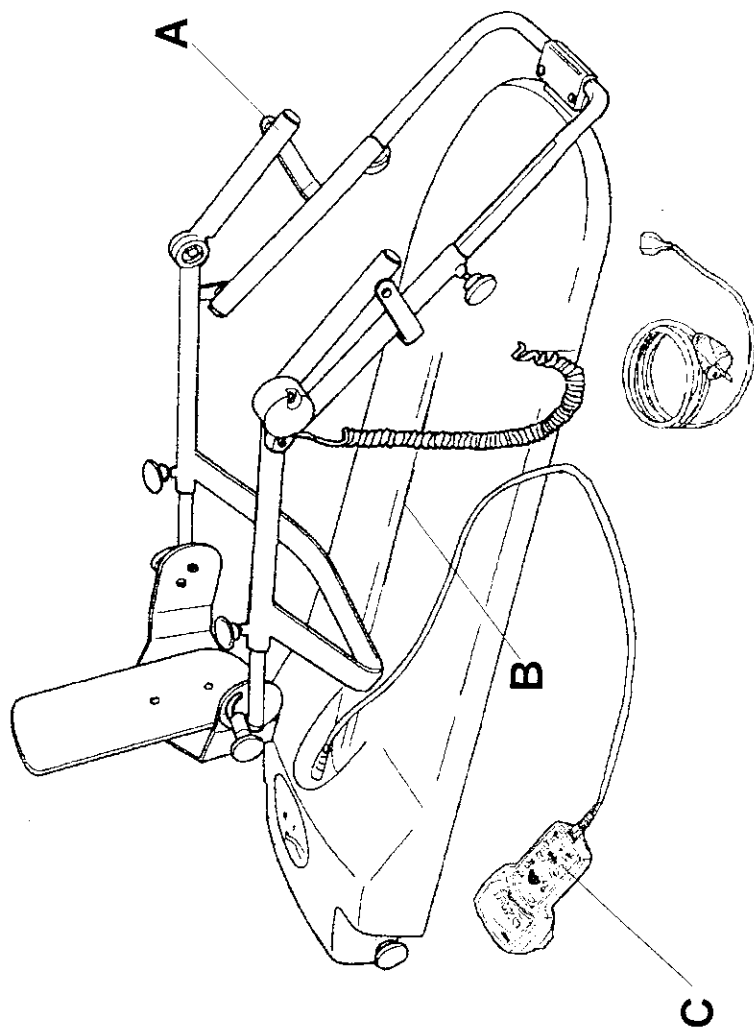Smith & Nephew Kinetec se réserve le droit de toutes modifications techniques.  
*Smith & Nephew Kinetec reserves the right to effect technical modifications.*

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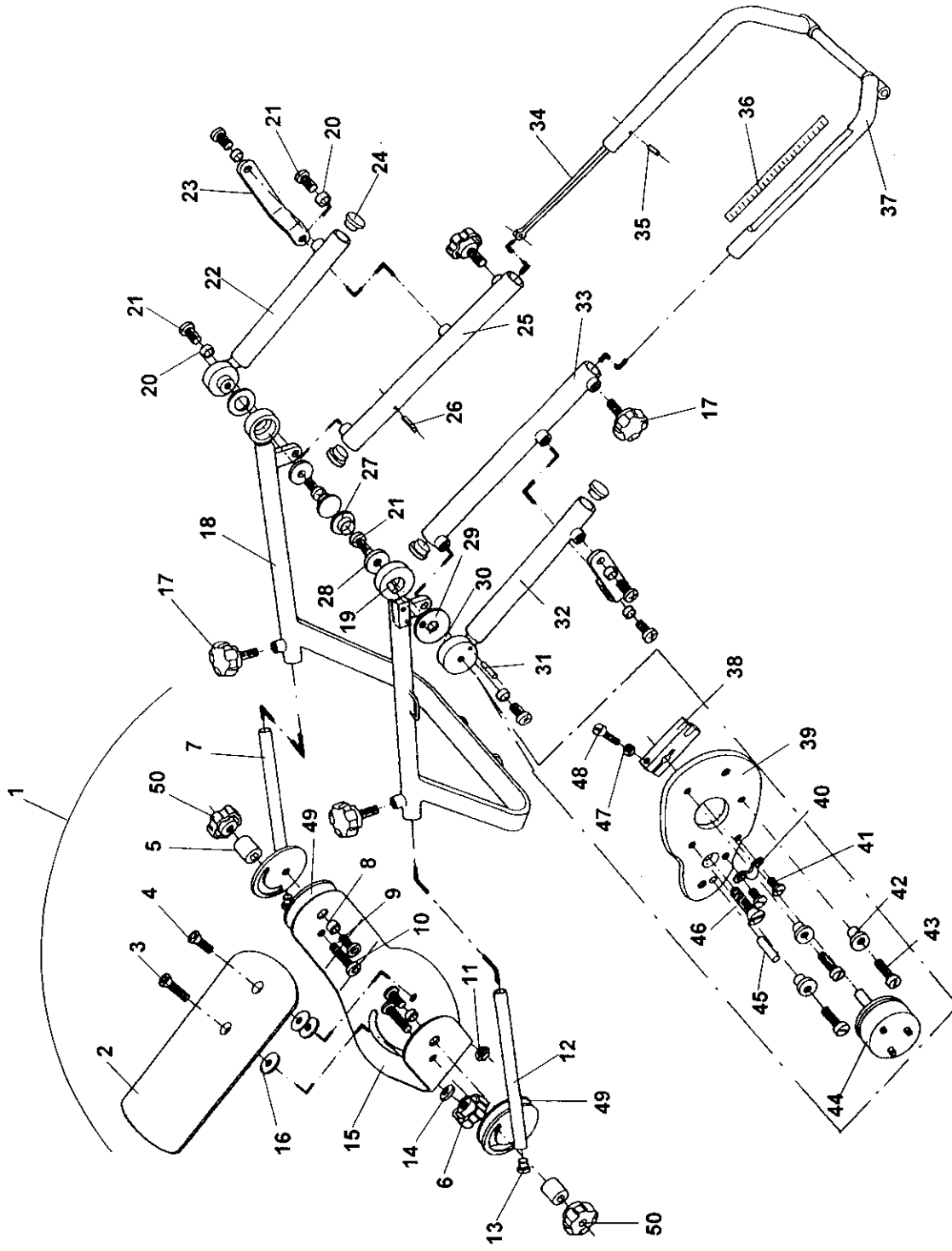
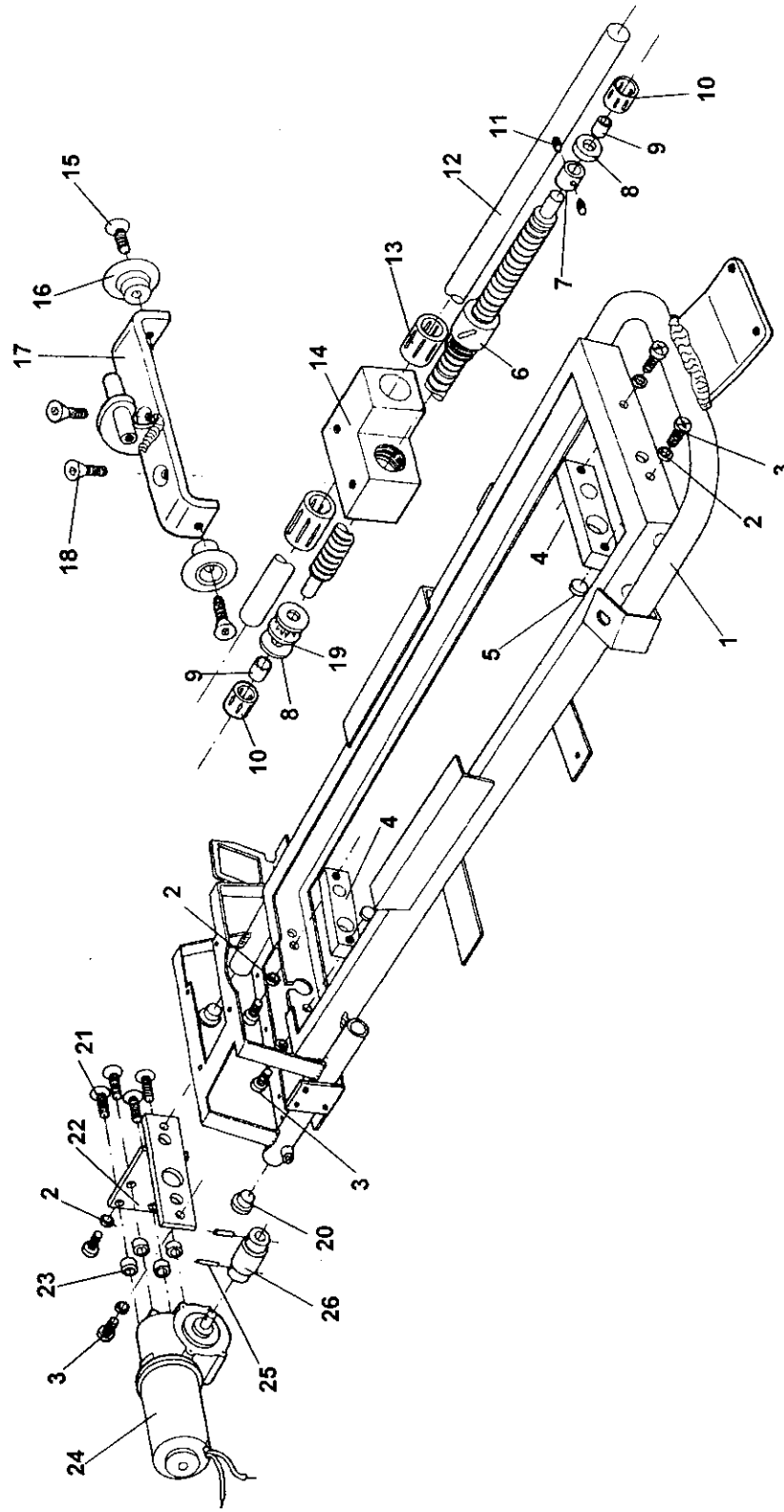
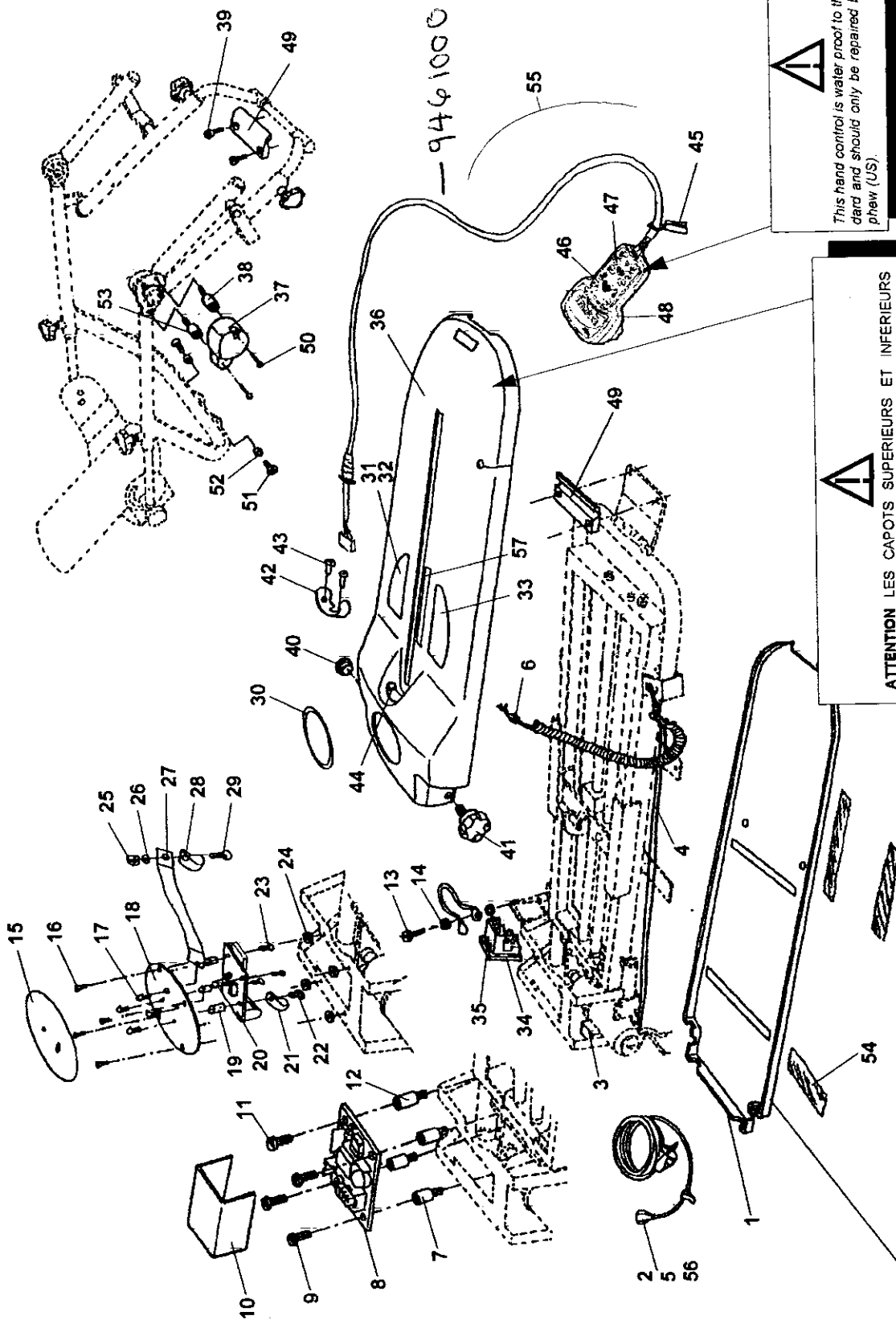
# ELECTRONICS

-94610006907

This hand control is water proof to the UL 544 standard and should only be repaired by Smith & Newphew (US).

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BE CAREFULL THE UPPER AND LOWER COVERS ARE ASSEMBLED BY DOUBLE SIDE ADHESIVE.




REF: 4635005562 - ADHESIF DOUBLE FACE Lg : 1 METRE - DOUBLE SIDE ADHESIVE Lg : 1 METRE

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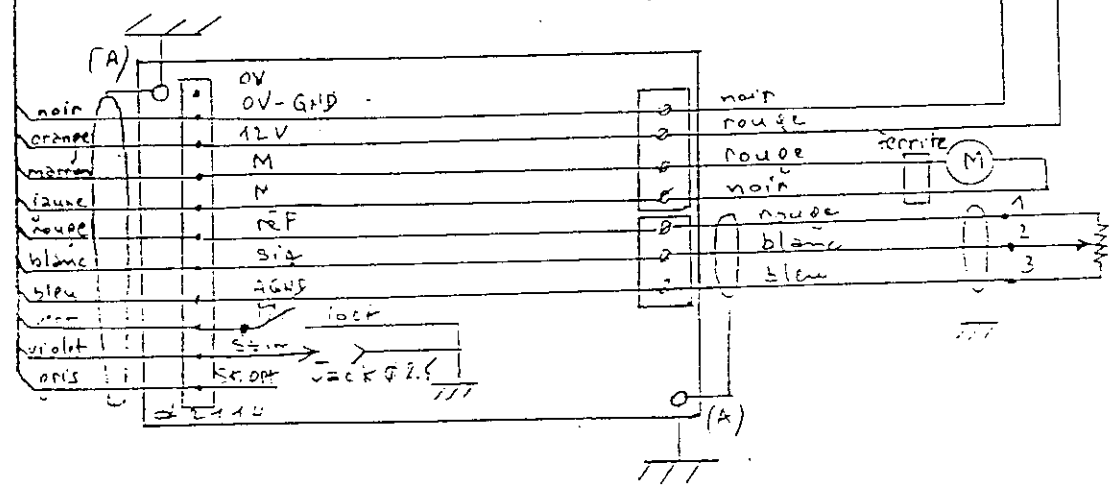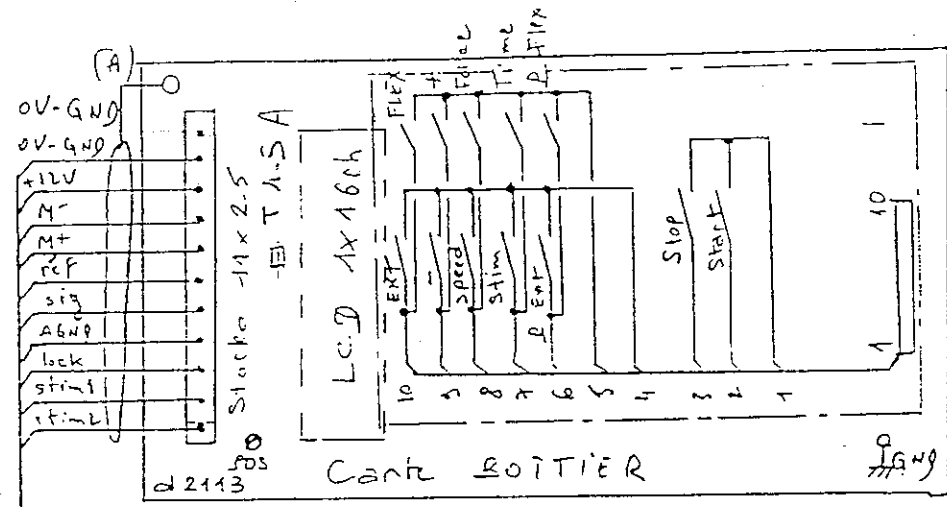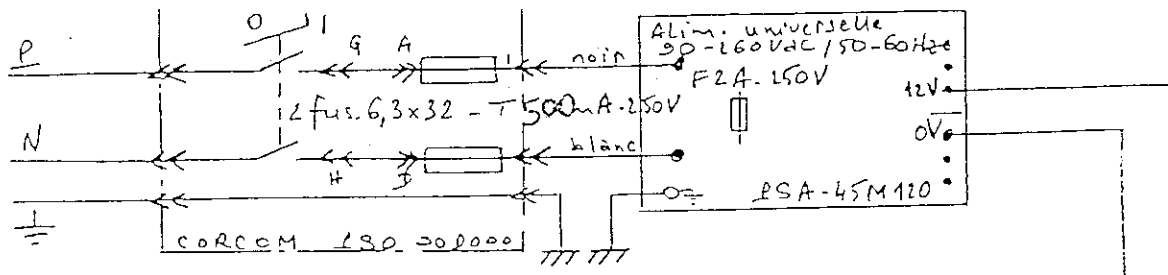# • User manual • KNEE

Before use, please read this document.

Smith & Nephew Kinetec reserves the right to effect technical modifications.

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14.7 Chariot de rangement

## 1 • DESCRIPTION

The KINETEC *Optima* is a Continuous Passive Motion exerciser for EXTENSION-FLEXION mobilization of the HIP and KNEE joints.  
A - Lower limb support • B-Thigh support • C-Foot plate • D - Hand control • E - Power cord

## 2 • ELECTRICAL CONNECTION: SAFETY FIRST.

The KINETEC *Optima* is a Type B Class I machine.

Plug the power cord (1) on the KINETEC *Optima* unit into a receptacle (Voltage between 100 to 240 Vac 50/60 Hz).

**IMPORTANT:**

Check to be sure that the plug fits the socket being used.

The socket must be grounded.

Use only the power cord delivered with the unit.

Ensure that the power cords are not entangled in the machine during operation.

### Fuse change

Warning: Replace the fuse with one of the same type and value.

- on main socket, 2 fuses T 500 mA 250V (6,3x32)
- on power supply PCB, 1 fuse F 2 A 250V (5x20)
- on PCB, 1 fuse T 1.50 A

## 3 • SAFETY

The healthcare professional determines the CPM protocol to be used (set up, duration and frequency of each session).

The patient must understand the Start/Stop/Reverse function of the hand control.

The KINETEC *Optima* unit is CE marked (Directive 89/336/CEE)

**DANGER:** Explosion risk if used with flammable anesthetics.

## 4 • START UP

Turn on the power switch (2). The hand control will read **KINETEC OPTIMA**

**Locking the hand control settings: switch (3)**

**UNLOCKED POSITION**  
adjustments are possible.

**LOCKED POSITION**  
the OPERATIONAL settings can be read and the Start/Stop/Reverse function operated.

**Double locking:** simultaneously press keys PAUSE EXT and PAUSE FLEX for blocked the hand control, the display reads "LOCK", for unblocked the hand control, simultaneously press the same keys, the display reads UNLOCK".

We recommend that the switch be kept in the locked position when the hand control is given to the patient.

### Display

4 - 3 characters indicate the EXTENSION limit.

5 - 3 characters indicate the FLEXION limit.

6 - 8 characters indicate messages. (RUN, STOP, EXT, FLEX, SPEED, FORCE, PAUSE, etc.).

7 - 3 characters indicate the real angle of the KINETEC *Optima*.

### Key functions

8 - Reading or selecting EXTENSION limit from - 5° to 110°.

9 - Reading or selecting FLEXION limit from 115° to 0°.

10 - Reading or selecting TIME COUNTER from 24H00MIN to 00H00MIN

At the end of the session, the display reads **TIMER 00H00MIN** and the unit stops at an average angle use.  
Depress + or - to select a new session time.

11 - These keys enable you:

- while selecting, to reduce or increase the operating values (blink value),
- while moving, to exceed the EXTENSION or FLEXION limits (BY-PASS MODE).
- while stopped, to manually move to EXTENSION or FLEXION (MANUAL MODE).

To activate BY-PASS MODE or MANUAL MODE, press the key for at least 3 seconds

Press the EXTENSION or FLEXION key to memorize the true angle reached, if necessary.

12 - This key enables you to start.

12' - This key enables you stop movement, press START again to reverse movement.

**IMPORTANT:** for maximum safety, the hand control should always be given to the patient.

13 - Reading or selecting SPEED from 5 to 1.

14 - Reading or selecting SYNCHRO-STIMULATOR (please request the instruction sheet from your KINETEC distributor).

15 - Plug for SYNCHRO-STIMULATOR

(use only approved stimulators to EN 601.2.10 / 1987 standard and/or with EC marked).

16 - Reading or selecting movement inversion LOAD from: ■■■■ = maxi load to ■ = mini load.

17 - Reading or selecting PAUSE in EXTENSION or FLEXION limit from 900 seconds (15 minutes) to 0 second.

- RESET TIME function

The machine is at rest. Simultaneously press keys EXTENSION and FLEXION.

This function enables you to read the operation time since the last reset.

Press the key START twice to reset the counter.

## 5 • SPECIFICATIONS

### Product

Weight: 13 Kg (28.6 lbs)

Dimensions: 101cm x 33cm x 33cm  
(39.7 x 13 x 13 inches)

Range of motion: from -5° to 115°

Speed: from 30 to 160°/minute for average femur length

Patient sizing: from 1.40m to 1.90m  
(4.5 to 6.25 feet)

### Electrical

Power supply: from 100 to 240 V~

Frequency: 50/60Hz

Electrical power: 50VA

Type B Class I device

### Environmental

- Storage / shipping conditions:

Ambient temperature: -40°C to +70°C.  
(-40°F to +160°F)

Relative humidity up to 90%.

- Operating conditions:

Ambient temperature: +10°C to +40°C.  
(+50°F to +105°F)

Relative humidity 80%.

The KINETEC HYGIENIC pads are designed for rapid fitting, optimal hygiene and maximum patient comfort.

- For using and positioning the straps, please refer to drawing number 6. Make sure that the self-adhesive parts (18) are visible.
- Place the sponge side next to the skin.

**FOR OPTIMAL HYGIENE, A NEW SET OF PADS SHOULD BE USED FOR EACH PATIENT.**

Each cover is provided with a label to record the patient's name.

#### CLEANING:

- The cover can be autoclaved at 125°C (225°F).
  - The straps and the foot pad are machine-washable (30°C, 90°F) with chlorine bleach.
- Part number to order the complete set : 4650001058

## 7 • SETTING UP THE PATIENT

- Make sure all knobs are tight.
- Place the KINETEC Optima machine in a position that will be comfortable for the patient.
- Measure in cm or inches the length of the patient's femur (L); adjust the thigh support to this measurement using knob (19).
- To install the patient on the KINETEC Optima machine, push the foot plate (C) up to the patient and tighten the knob (20).

#### IMPORTANT

Adjust the axis of the patient's hip (21) with the axis rotation (22) of the KINETEC Optima machine, and the axis of the patient's knee (23) with the axis rotation (24) of the KINETEC Optima machine.

## 8 • POSITIONING THE FOOT PLATE

The articulated foot plate allows the positioning of the foot from 40° of PLANTAR FLEXION to 30° of DORSIFLEXION and the leg in 30° of EXTERNAL or INTERNAL ROTATION.

- The knob (25) allows positioning the foot in plantar flexion or dorsiflexion.
- The knob (26) allows positioning the leg in external or internal rotation.

## 9 • MAINTENANCE

After 2,000 hours of working time, the KINETEC Optima machine needs to be lubricated (joints, ball bearing and threaded rod). The message SERVICE !! on the hand control when the power on indicates that the service interval has been reached.

You can continue use your KINETEC machine by pressing the START key, but you must contact your KINETEC specialist to undertake this maintenance operation.

## 10 • BREAKDOWN

- If you need a spare parts catalogue and/or service manual, please request it from your KINETEC distributor.
- If, after connecting the power cord on the KINETEC Optima machine and switching the machine ON (2), the display indicates no information:
  - Try another electric device to make sure that the power supply is uninterrupted.
  - Change the fuse of the socket (27) using a fuse of the same type and value: 2 fuses T 500 mA 250V (6.3x32mm).
  - If the display does not indicate any information after following these steps, contact your KINETEC specialist.
- If, after switching on the power to the KINETEC Optima using the switch (2), your machine fails to operate and the display indicates 15 STOP 25 115, press the START key once again.
- If your machine still fails to operate, contact your KINETEC specialist.
- If any of the following messages persists after switching the machine off and pressing the START key, contact your KINETEC specialist:
  - SERVICE D1 (failure of measuring angle)
  - SERVICE D2 (failure during motion)
  - SERVICE D3 (failure of motor direction).

## 11 • CLEANING

First, switch the CPM unit OFF and disconnect the power cord from the receptacle.

Spray a DISINFECTANT product (i.e., PROPANOL/ISOPROPANOL or ALDEHYDE solution) on to the SURFACES (plastic leg support, plastic covers and metal parts).

To clean the KINETEC hygienic pads, please refer to paragraph 6.

## 12 • SYMBOLS

|                                                                                     |                                                        |
|-------------------------------------------------------------------------------------|--------------------------------------------------------|
|  | TYPE B EQUIPMENT<br>(insulated against electric shock) |
|  | WARNING<br>(consult accompanying documents)            |
|  | OFF (disconnected from the mains)                      |
|  | ON (connected to the mains)                            |

|                                                                                      |                     |                                                                                       |           |
|--------------------------------------------------------------------------------------|---------------------|---------------------------------------------------------------------------------------|-----------|
|   | Hand control unlock |                                                                                       |           |
|   | Hand control lock   |                                                                                       |           |
|  | FLEXION             |  | EXTENSION |
|  | Stimulator          |                                                                                       |           |

## 13 • GUARANTEE LIMITS

The KINETEC guarantee is strictly limited to free replacement or to repair of the part or parts acknowledged as defective, at the Smith & Nephew factory.

Smith & Nephew Kinetic guarantees its passive motion device against any manufacturing defect for 1 year from the date of purchase by the consumer.

Smith & Nephew Kinetic alone is qualified to judge whether the guarantee applies to the specific problem.

The guarantee cannot be applied if the apparatus has been subjected to abnormal use or has been used in circumstances other than those stipulated in our user manual.

The guarantee will not be applied in case of deterioration or accident due to negligence, lack of supervision or maintenance or an attempt to repair the equipment.

## 14 • ACCESSORIES

- |                          |                                        |
|--------------------------|----------------------------------------|
| 14.1 Plastic leg support | 14.5 Clamp for the foot-end of the bed |
| 14.2 Bed straps          | 14.6 PT table clamp attachment         |
| 14.3 Wheels              | 14.7 Cart                              |
| 14.4 Carrying handle     |                                        |

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*We*

**Smith & Nephew Kinetec**

(name of manufacturer or his representative established within the European Community)

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*declare on our own responsibility that product*

### **KNEE KINETEC model OPTIMA**

(brand name, model, (N° of lot or series if necessary))

*complies with the assesement criteria of Council's Directives:*

- n° 93/42/CEE of June 14, 1993 - MEDICAL DEVICES  
(Annex II)  
mandatory date : June 14, 1998
- n° 89/336/CEE of May 3, 1989 - ELECTROMAGNETIC COMPATIBILITY  
mandatory date : January 1, 1996

*and the quality system of the manufacturer is in conformity with the following standard (s) :*

**EN 46001/96 - ISO 9001/94**

Certificate n° 0214/46001/9001 established by Gmed on 15/10/97

**Certificate for approval of full Quality Assurance System - Annexe II.3 -  
N° 0214/BP3/1 established by Gmed**

*Additional information :*

**Product file : OPTIMA dated 5/06/97**

*Year in which CE mark was affixed : 1997*

Tournes, October 20, 1997  
(place and date)

**P. SONNET**  
General Manager

(Name, function and sign of authorized people)

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Certificate 2/3 N° : 0214 / B2P3 / 1

Delivered at Fontenay-aux-Roses on November 12<sup>TH</sup>, 1997

# EC CERTIFICATE

APPROVAL OF FULL QUALITY ASSURANCE SYSTEM  
ANNEX II point 3 DIRECTIVE 93/42/EEC concerning medical devices

Device(s) category :

Immobilization, mobilization and testing devices  
of the human joints

Identification of device(s) :

See EC declaration of conformity of  
the manufacturer authenticated by G-MED

Manufacturer(name and address) :

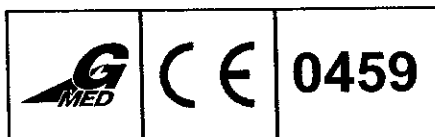
SMITH & NEPHEW - KINETEC S.A.  
Zone Industrielle de Tournes - Cliron  
BP 19  
F - 08090 TOURNES

E.U. Responsible :

SMITH & NEPHEW - KINETEC S.A.  
Zone Industrielle de Tournes - Cliron  
BP 19  
F - 08090 TOURNES

The G-MED certifies that, on the basis of the results contained in the file referenced 30273304, the quality assurance system - for design, production and final inspection - of medical devices designated above is in conformance with the requirements of the directive 93/42/EEC, annex II point 3.

This certificate is valid until : November 12<sup>TH</sup>, 2002 (included)

E. GRIMAUD  
G-MED General Manager

In compliance with clause 17, CE marking shall be accompanied by our identification number : 0459  
This certificate is issued under the following conditions :

- 1 It is valid only to the designated device(s) listed above.
- 2 Copies are available upon request by the manufacturer.

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Certificate 1/3 N° : 0214 / 46001 / 9001 / 1

Delivered at Fontenay-aux-Roses on October 15<sup>TH</sup>, 1997

# CERTIFICATE EN 46001 / ISO 9001

## APPROVAL OF QUALITY ASSURANCE SYSTEM

**Scope of certification :** Design, manufacturing, sales and servicing of immobilization mobilization and testing devices of the human joints

**Delivered to :** SMITH & NEPHEW - KINETEC S.A.

**For the site(s) of :** Zone Industrielle de Tournes - Cliron  
BP 19  
F - 08090 TOURNES

The G-MED certifies that, on the basis of the results contained in the file referenced 30273304, and according to the scope of certification, the system of quality assurance is in conformance with the requirements of standard EN 46001 : 1996 and ISO 9001 : 1994.

This certificate is valid until : October 15<sup>TH</sup>, 2002 (included)

**SYSTEM CONFORMITY**

**E. GRIMAUD**  
G-MED General Manager

This certificate is issued under the following conditions :

- 1 It is valid only to the scope of certification listed above.
- 2 Copies are available upon request by the manufacturer.

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