
















Program Mode

1. Press OPTS  until PROGRAM MODE is displayed
2. Press ON .
3. Enter the rate for Step 1 and press OK .
4. Enter the volume for Step 1 and press OK .
5. Repeat Steps 3 and 4 until all steps are complete and press OK .
6. Review the Total VTBD and press OK .
7. Press START .

Ramp Taper Mode

1. Press OPTS  until RAMP TAPER MODE is displayed
2. Press ON .
3. Enter the total bag volume to be delivered and press OK .
4. Enter the Ramp Up time in hours and minutes and press OK .
5. Enter the Taper Down time in hours and minutes and press OK .
6. Enter the Total Time for the infusion in hours and minutes and press OK .
7. Review the calculated Max Rate and press OK .
8. Press START .

Alarm Troubleshooting

Air In Line!

- Check for air in air-in-line detector.
- Check that tubing is properly inserted in air-in-line detector.
- Remove air from line.

Battery Too Low!

- Plug pump into wall outlet.


Door Open!


- Close door.

Dnstream Occlusion!

- Open clamps.
- Check for kinked tubing.
- Check IV site.
- Check occlusion pressure setting.

Pump Stopped!

- Pump is stopped.
- Either start infusion or press  to extend hold time 3 minutes.

Silence alarms by pressing .

Refer to Instructions for Use for complete operating instructions.

B|BRAUN

B. Braun Medical Inc.
824 Twelfth Avenue
Bethlehem, PA 18018
1-800-227-2862
www.bbraunusa.com

Vista™ basic

Infusion Pump



Quick
Reference



B|BRAUN

Rx only
Vista™ and Working Together for Excellence in Safety
are a trademarks of B. Braun Medical Inc.
©2002, B. Braun Medical Inc.

950853-Rev B 06/02


Important Keys

Yes, No, OK, etc.

Number Keys	7	8	9	0		On/Off (Power)
	4	5	6	C		Alarm Silence
	1	2	3	■		Start/Stop
					Alarm Light Infusing Light	

Setting the Pump

- Press (On/Off).
- Press button on top of door to open.
- Insert tubing into pump.
 - Push open tube pincher **A**
 - Place tubing in tube pincher **A**
 - Insert tubing along channel **B**
 - Push tubing into Air In Line Detector **C**

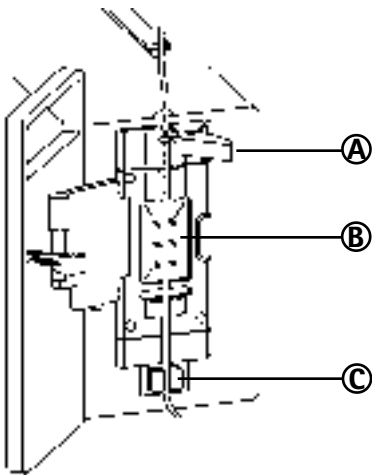


- Close Door.

- Is this a new therapy?
 - Yes : clears current infusion data.
 - No : returns pump to last programmed infusion.
- Prime set?
 - Yes : follow prompts to prime tubing.
 - No : begin programming infusion.

Continuous Mode


- Enter the rate using the number keys.

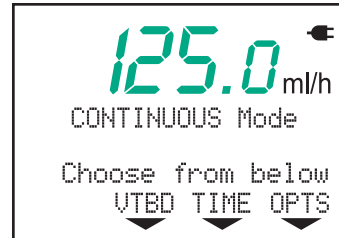

- Press VTBD .
- Enter the bag volume using the number keys.
- Press OK .



- Press START .

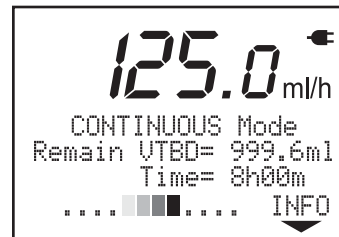


Piggyback Mode

- Press OPTS until PIGGYBACK is displayed
- Press ON .
- Press END .
- Press PIGY .
- Enter Rate/VTBD or VTBD/Time as in Continuous Mode.
- Press START .

Changing the Rate

- While infusing, enter the new rate using the number keys.
- Press OK .

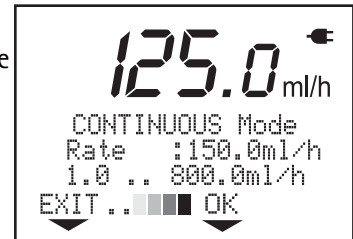

NOTE:

If the OK key is not pressed within 10 seconds of the change, the infusion will continue at the original rate.

Changing the VTBD

- Stop infusion.
- Press VTBD .
- Press **C**.
- Enter the new volume to be delivered.
- Press OK .
- Press .

